

GraviBody®

Support and compression garment to relieve pregnancy related discomfort in the back as well as pelvic and vaginal pressure.

Indications:

- Preventive function for weakness of the connective tissue, vulva varices, previous complications in former pregnancies and twin pregnancy
- Therapeutic function for pain in the back, hip and symphysis area
- Post-partum for symphysis relaxation

Instructions for Use:

Take hold of edges of GraviBody®'s fuselage fabric and hold in front of you so that Velcro of perineum belt points forward (recognizable by four blossoms, cf. fig.). Coevally clasp the edges of shoulder belts and stomach belt on the right and left side. Put on GraviBody® like trousers. Pull straps over shoulders and arrange buckle that binds the straps between your breasts. Straps must not be twisted. Adjust desired strap tension for utmost wearing comfort and ideal support. Possible with: straps, perineum belt in the front (Velcro), perineum straps in the back, stomach belt on the right and left side (Velcro). A second perineum belt is enclosed.

Wear over absorbent underwear.

GraviBody® is available in the female standard sizes 36 to 46. We will manufacture special sizes by request.

Care Instructions: Hand wash only. Do not iron. Do not bleach. Do not use dryer. Let dry off in wet condition.

GraviBody® is manufactured in accordance to the strictest German quality standards. Carefully selected textiles, which have been checked for safety according to Öko-Tex Standard 100 as well as supreme production quality guarantee your satisfaction.



GraviBody® in front and back view

We appreciate your purchase of GraviBody®. We hope the product is to your satisfaction.

Wishing best progression of your pregnancy,

Tomed Dr. Toussaint GmbH
Medizinische Produkte
Lindberghstr. 3A
D-64625 Bensheim
www.tomed.com

Date of information: 9 November 2008



The product meets the requirements of Council Directive 93/42/EEC from June 14 1993 for medical products